



USER GUIDE

DIGITAL STETHOSCOPE





With Omply's digital stethoscope, listening to heart, lung and bowel sounds at home is easy and safe. Listening with the stethoscope can detect normal sounds as well as, in connection with illness, signs such as wheezing and crackles in the breathing or heart murmurs and arrhythmias. The sounds are transmitted automatically to the app, and your doctor hears them in real time.

Connecting the stethoscope to your phone

- Turn on the stethoscope by pressing and holding the power button for a few seconds.
- The device connects automatically to your phone and the app.
- If you wish, connect your own headphones to the device (AUX connector) to hear the heart and lung sounds yourself.
- Handle the stethoscope gently.
- Grip the black upper part of the device firmly with all five fingers.
- Press the blue listening end firmly against the desired listening point and avoid moving your fingers during listening.



During and after the examination

- Your doctor will always guide you on where to place the stethoscope and when to change position.
- When listening is finished, turn off the device by pressing the power button until a shutdown notification appears on the screen.
- The stethoscope runs on a rechargeable battery and can be charged with the included Micro-USB cable.

